MD.FLOW

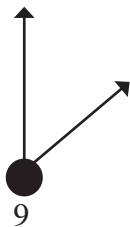
COMPRESSOR
NEBULIZER SYSTEM

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- 7- **3TM MD.FLOW** Nebulizer
- 8- Air tube
- 9- Mask adult+ mask child

TECHNICAL SPECIFICATIONS

- 1- Modal: **3TM MD.FLOW**
- 2- Electricity supply :220v/60hz
- 3- Nebulizer air flow: 3.31 l/min
- 4- Compressor air flow: 7.5 l/min
- 5- Compressor max pressure: 3.0 bar
- 6- Weight: 2 kg
- 7- Size: (H) 95 (L) 175 (w) 160 mm
- 8- Noise level: Less than 55dBA
- 9- Nebulization rate: Ca.0.2 ml/min
- 10- Particle size: MMAD 1.8 urn (nacl)
- 11- Nebulizer capacity: 8 ml

* Note:

please register your factory information about this device (unit)

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WARRANTY CONDITIONS

- 1- The device is covered by a 5 year warranty form date of purchase for any defects in the design the materials used.
- 2- the warranty consists in the replacement & or repair free of charge.

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3. The warranty does not cover the accessories with the device and those parts subject to normal wear and tear.

4. The device must be repaired by authorized technical service centers only.

5. The device must be sent to the authorized service centre for repair within 8 days after the defect is noticed.

6. The transport costs for the device are at user's charge.

7. Any repair out of the warranty conditions are at user's charge.

8. The warranty does not cover failures resulting from misuse, abuse of the product or in case the damage is not due to manufacturer's liability (accidental fall, rough transportation,etc.)

9. The warranty does not involve any compensation for any kind of damages to persons or things, direct or

Indirect, occurred when the device is not operating.

10. The warranty is valid from the date of purchase certified by the ticket or the invoice which have to be peremptorily attached to the warranty coupon.

11. The lack of warranty coupon properly filled in and validated by a certification of purchase voids the warranty.

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IMPORTANT:

Read these instructions carefully before using this device.

INSTRUCTIONS FOR USE

REMOVE the compressor from carrying case before use check the unit before each use.in order to check for possible function anomalies and / or storage. when inhaling. sit upright and relaxed at a table and not in armchair as otherwise you will constrict your respiratory passages and impair the effect inhalation.

- 1- As soon as you have unpacked the device. check it for visible damages or defects. In particular. check for cracks in plastic housing. which could expose electrical components.
- 2- check for accessories integrity.
- 3- Plug the device into the wall socket. Making sure the electrical rating shown on the rating plate on the Bottom of the unit corresponds to your main voltage.
- 4- To start the treatment. switch the device on by pressing the ON/OFF switch
- 5- Inhale the aerosol solution using the prescribed accessory

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6- When the treatment has been completed. switch the unit off by setting the switch position O and Disconnect the plug from the wall socket.

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7. Wash the nebulizer and accessories as described in CLEANING AND DISINFECTION OF THE ACCESSORIES section.

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8. The unit when operating must be placed on stable and horizontal surface.

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9. Make sure the air vent opening are not obstructed.

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10. Do not leave the unit plugged when not in use.

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